


## What Happens During the FemVue Test?

First, transvaginal ultrasound is used to look at your anatomy in more detail. Then, a thin balloon catheter is placed into your uterine cavity and FemVue is used to start the fallopian tube test.

FemVue creates natural saline and air "bubbles" that are seen with ultrasound and allows your doctor to determine if your tubes are open or blocked.

The results of the test are available to you immediately and your doctor can discuss possible next steps within the same visit.



Natural saline & air "bubbles"

## FemVue is a reliable<sup>1</sup> & convenient test

that allows your doctor to quickly check if your tubes are open or blocked in their office.

### Frequently Asked Questions

#### When is the test performed?

Your test will be scheduled during the first half of your menstrual cycle, before you ovulate, to reduce the chance of performing the test when you may be unknowingly pregnant.

#### Is the test uncomfortable?

You may experience mild or moderate uterine cramping during the test.

#### Will I be given anything for possible discomfort?

Your doctor may advise you to take medications used for menstrual cramps to reduce discomfort.

#### When can I expect to know the results of the test?

You will know your results during or right after the test.

#### Can I still get pregnant if my tubes are blocked?

Even if your tubes are blocked, there are ways to overcome this problem and still get pregnant. Consult with your doctor for recommendations and possible next steps.

1. Beverley, et al. (2018) Evaluation of Tubal Patency with a Saline-Air Device. Journal of Reproductive Medicine, 63(3):120-126.



# Do you have a **silent infertility factor?**



## Find out with FemVue

A diagnostic test to check your fallopian tubes

[www.FemVue.com](http://www.FemVue.com)



© 2019 Femasys Inc.

00926:01  
25.Jan.2019

## Why Should I Have My Tubes Evaluated?

Blocked fallopian tubes are usually a silent infertility factor. Approximately 1 out of 3 infertile women are living with blocked tubes and most have no symptoms.<sup>1</sup>

### Factors that can lead to blocked tubes:

- Previous pelvic infection
- Sexually transmitted diseases (STD)
- Prior surgery (abdominal)
- Prior ectopic pregnancy
- Endometriosis
- Unknown

The only way to know if this is the reason you're not getting pregnant is to check your tubes.



According to the American Society for Reproductive Medicine (ASRM), tubal disease is an important cause of infertility and should be specifically excluded.<sup>2</sup>

1. Beverley, et al. (2018) Evaluation of Tubal Patency with a Saline-Air Device. Journal of Reproductive Medicine; 63(3):120-126.

2. The Practice Committee of the American Society for Reproductive Medicine. Optimal evaluation of the infertile female. Fertility and Sterility. 2015;103:e47.

## FemVue Brings You One Step Closer

FemVue is a revolutionary device that allows your doctor to easily check your tubes as part of your initial infertility work-up.

### FemVue is...



#### Safe:

- Use natural saline and air
- No radiation exposure
- No iodine-based x-ray dye



#### Time Saving:

- Quick office test
- Real-time results
- Plan next steps immediately



#### Cost-Effective:

- More affordable than alternative tests
- Lower overall costs



#### Convenient:

- In your doctor's office
- Single visit



Get the answers you're looking for from the doctor you know and trust.